# Tele-Fit

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*Friction elements made of high elasticity acrylic*

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## Indication:

The Tele-Fit friction elements are used for telescopic and conus crowns to restore lost friction.

The friction elements are made of a very hard acrylic material.

## Processing:

The Tele-Fit can also be fitted into existing double crown jobs and can easily be exchanged.

Fig. 1

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*To make further processing easier, use the centre drill (Order No. 9884) to set a mark first.*


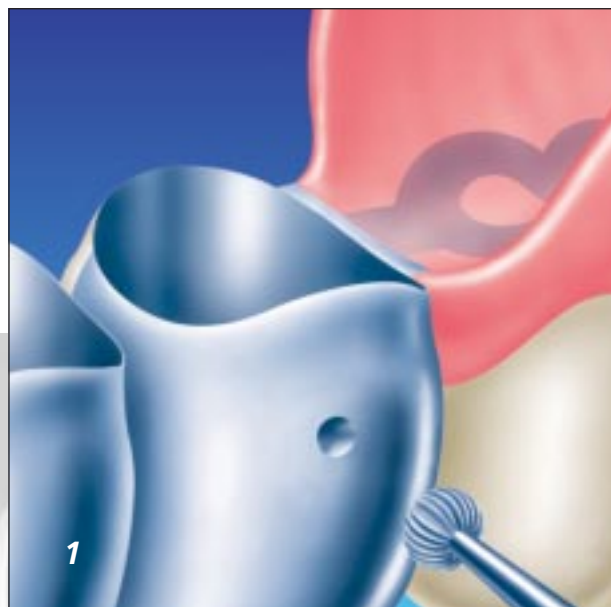
Fig. 2

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
*Then use the spiral drill to drill a hole through the crown. For precious alloys use Order No. 9883, for non precious Order No. 9885.*

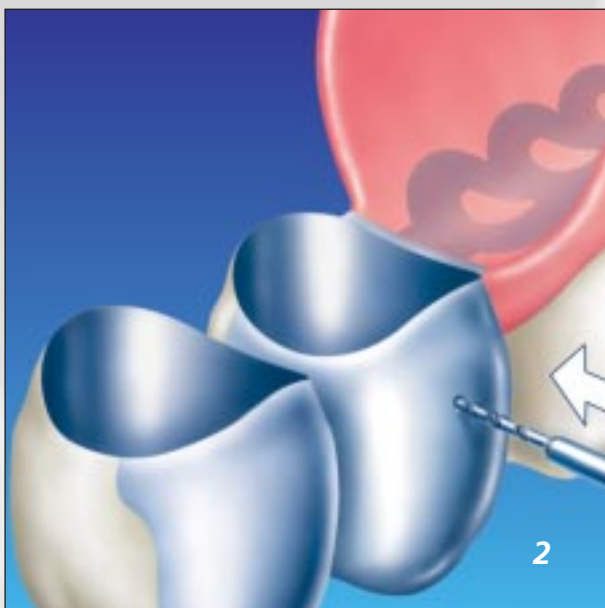**Fig. 3**

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*Now push the Tele-Fit into the hole as illustrated in fig.3.*

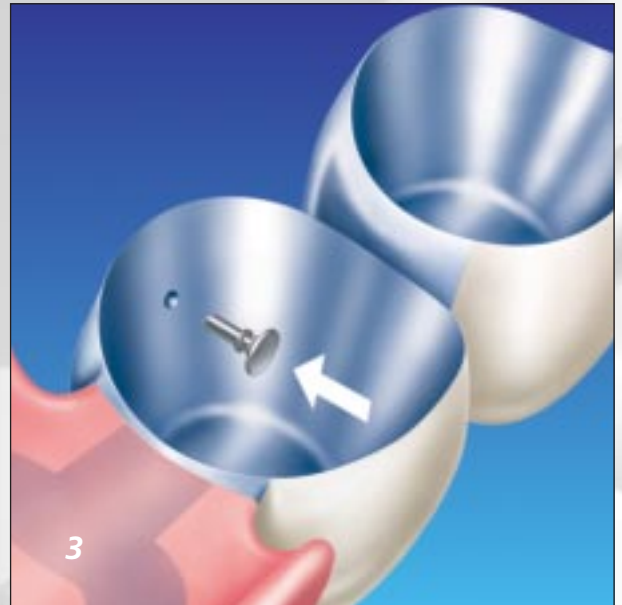**Fig. 4**

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

*The Tele-Fit is secured due to retention in the final position.*

**Fig. 5**

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*Using a Le Cron-instrument the Tele-Fit is pressed into it's final position.*

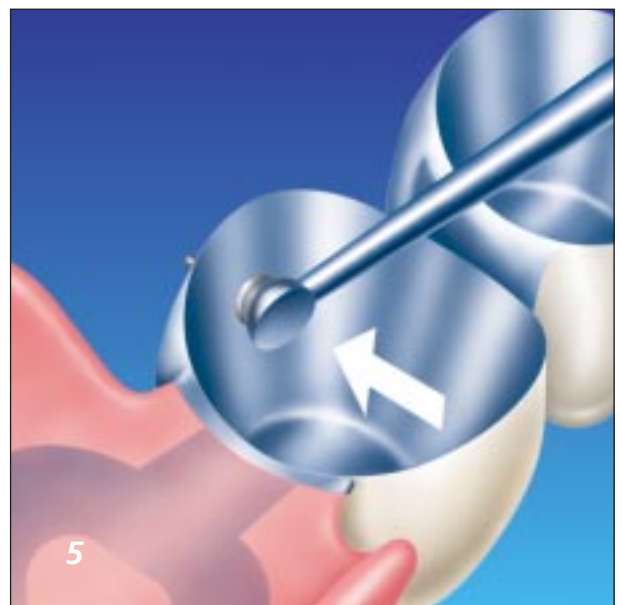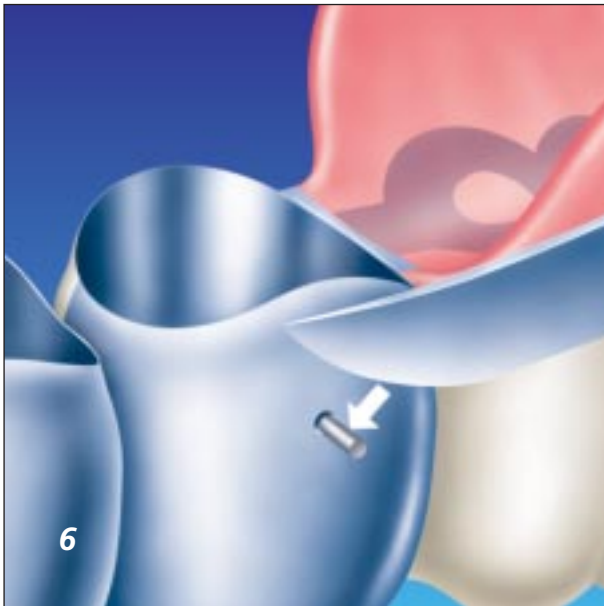
**Fig. 6**

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*The extending part of the Tele-Fit can be cut off flush with the crowns.*

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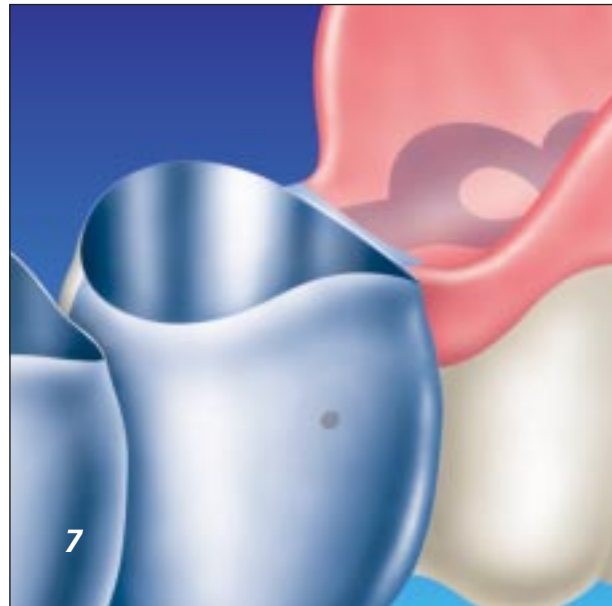*The Tele-Fit's finished situation.*

**Complete set**

- Order No. 9881** 6 x Tele-Fit white (0.10 mm)
- Order No. 9882** 6 x Tele-Fit beige (0.15 mm)
- Order No. 9883** Spiral drill
- Order No. 9884** Centre drill
- Order No. 9885** HM Spiral drill
- Order No. 9880** Starter set consisting of:  
10 Tele-Fit white,  
10 Tele-Fit beige,  
1 Spiral drill  
1 Centre drill

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Servo-Dental GmbH & Co. KG