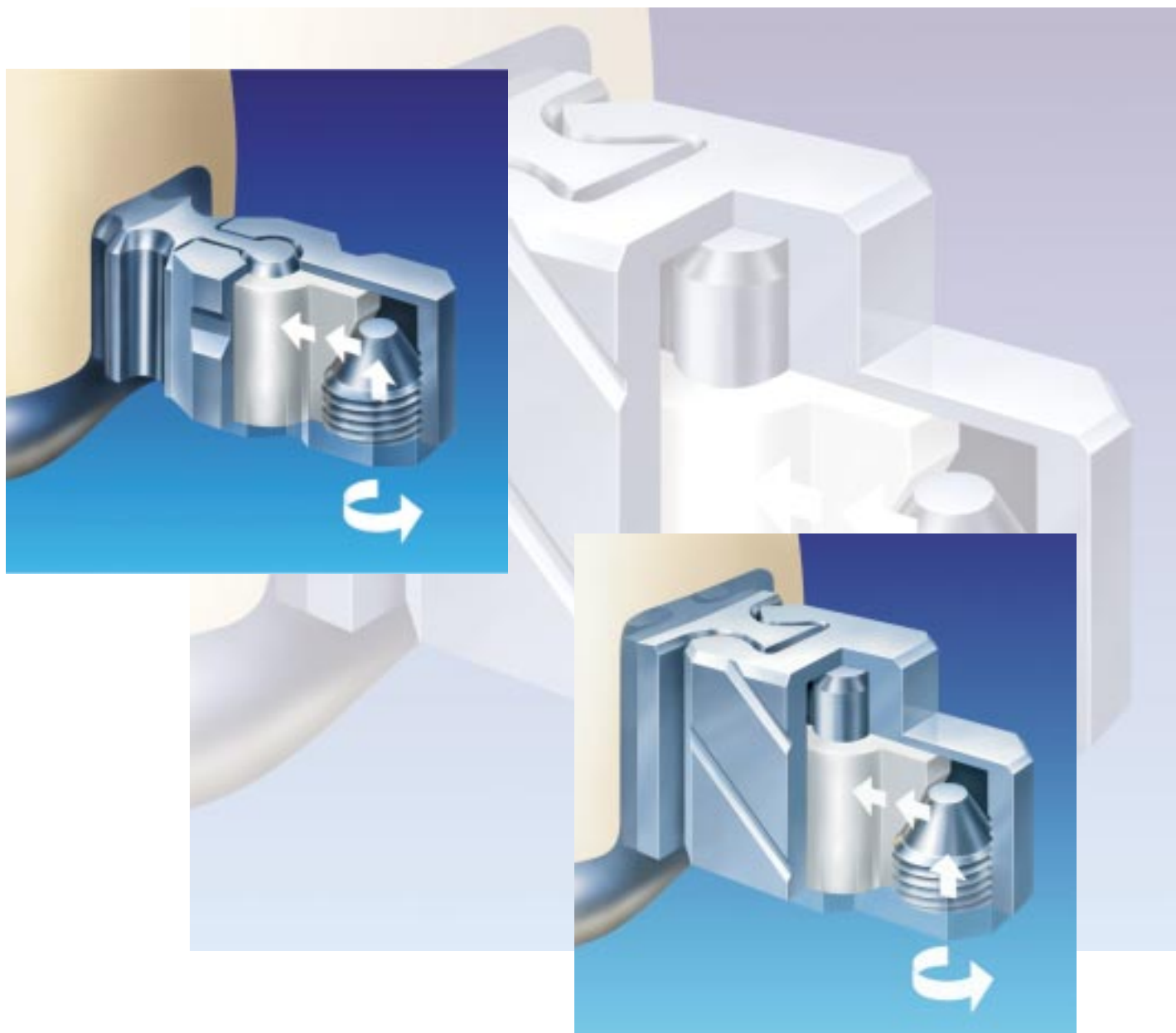
# **SD-Geschiebe / + LV** *SD-Attachment / + LV*

## SD-Geschiebe SD-Attachment

## SD-Geschiebe LV SD-Attachment LV

### Indikation:

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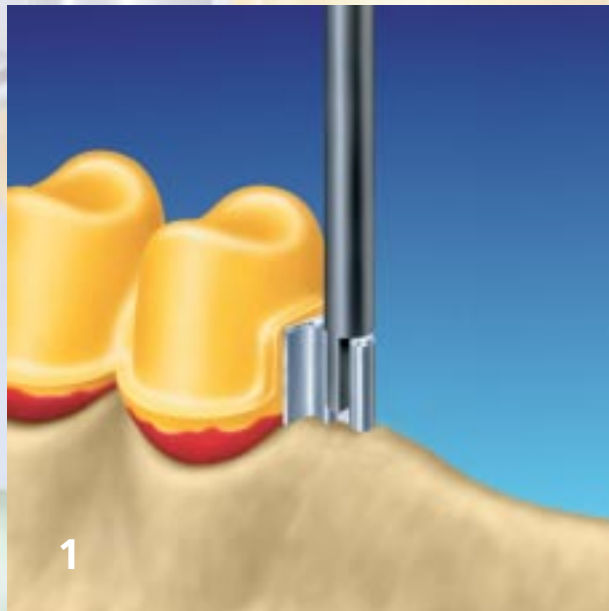
### Indication:



The Servo SD-attachments are vernier adjustable, extracoronary retaining elements. The SD-attachment-LV is a longer version of the classic SD-attachment and has integrated guidance in the housing, corresponding to the grooves in the male. The height of the SD-LV can be individually adjusted, depending on the situation. In both cases friction is achieved through vernier adjustment of the acrylic female sliding over the male. The male is either made of HEA or burn out acrylic and the housing and activating screw are both made of titanium. The housing can either be lasered or bonded with adhesives into the prosthesis.

### The particular advantages :

- height can be individually adjusted (LV)
- vernier adjustment of friction
- integrated guiding grooves (LV)
- simple exchange of the friction element

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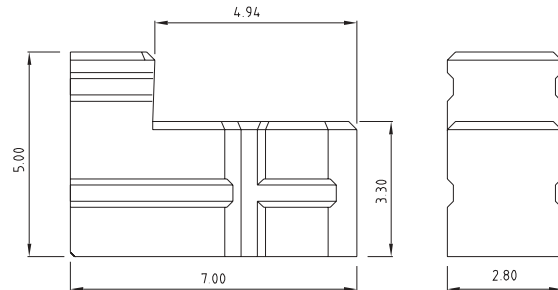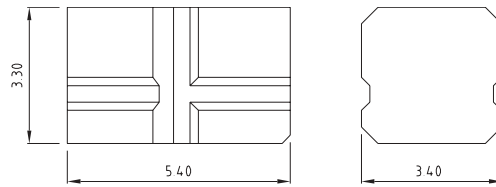
The SD-attachment must be positioned using the paralleling tool. Advantage: Optimum adaption to the jaw ridge. The SD-attachment should be used with a milling and interlock. The LV can be used without a milling, due to integrated guidance in the housing corresponding to the grooves in the male.


Fig. 2:

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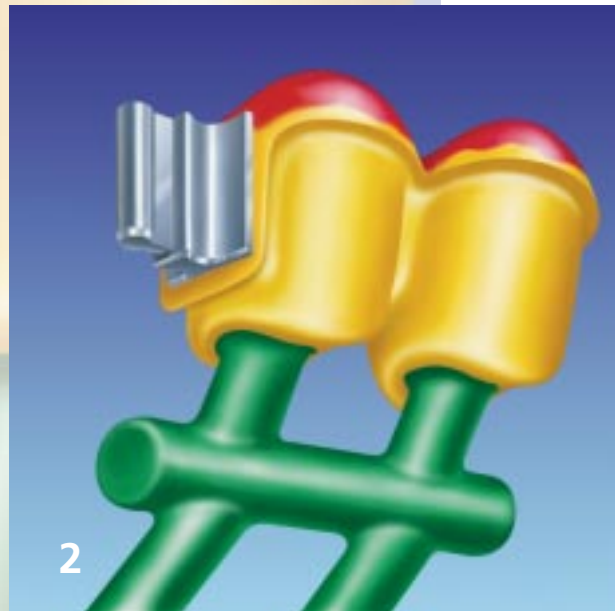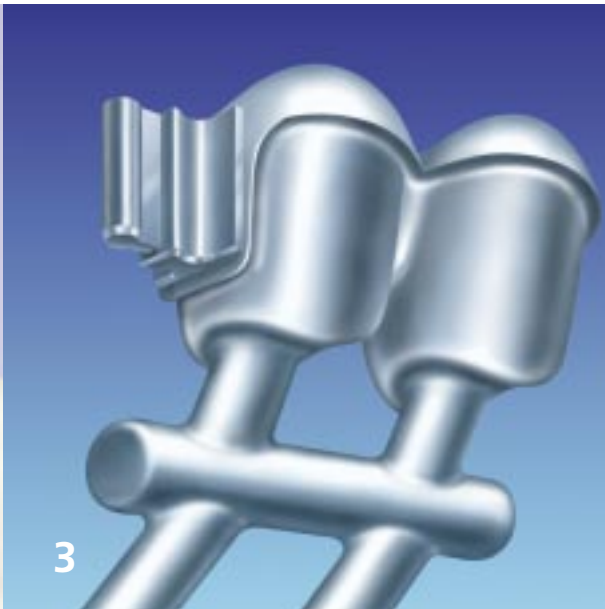
*While finishing the crowns take care when polishing the male so that there is little loss of the contour determination.*

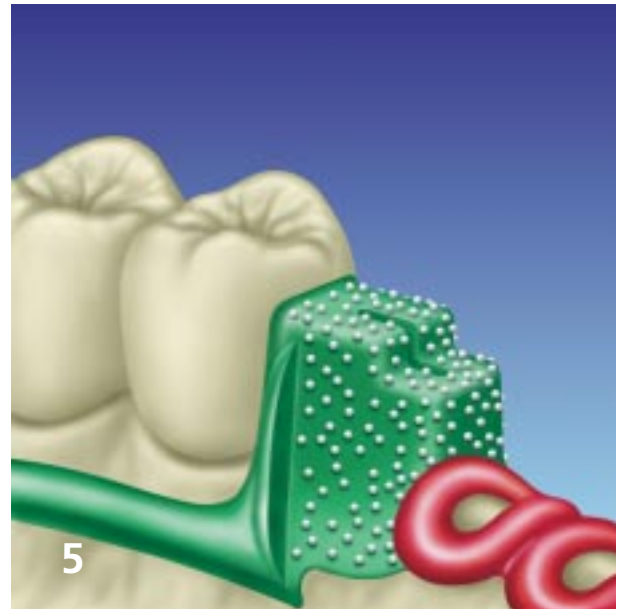**Fig. 4:**

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
*When finished fix red duplicating aid over the male and fill the undercuts with wax. The red duplicating aid is 0.02mm larger and is indicated whenever using adhesives. Duplicate as usual.*

**Fig. 5:**

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
*Wax the partial as usual leaving an excess opening at the top of the female housing so that adhesive can overflow. Invest, cast and finish as usual.*

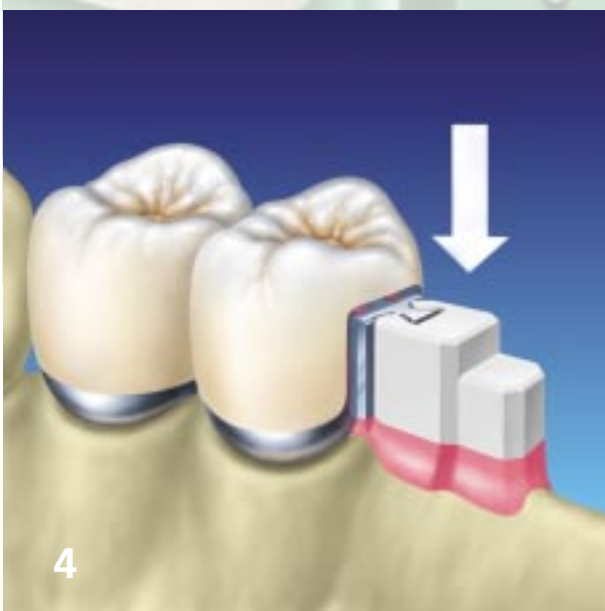**Fig. 6:**

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*Before bonding sandblast the inside of the partial using 250 µm aluminium oxide and then do not touch this area anymore.*

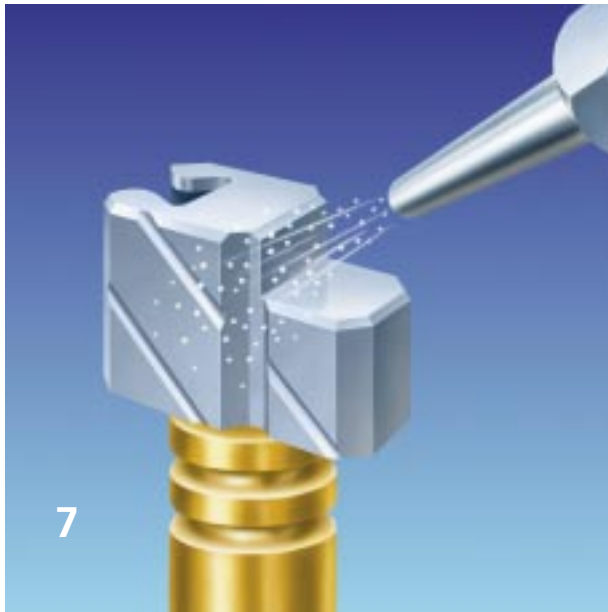**Fig. 7:**

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

*The female with housing is fixed onto the sandblasting aid and sandblasted with 110  $\mu\text{m}$  aluminium oxide and a pressure of 2 bar. Do not touch the sandblasted areas anymore.*

**Fig. 8:**

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*The female can now be set into position using the transfer tool. Undercuts and the opening where the male protrudes are filled with wax.*

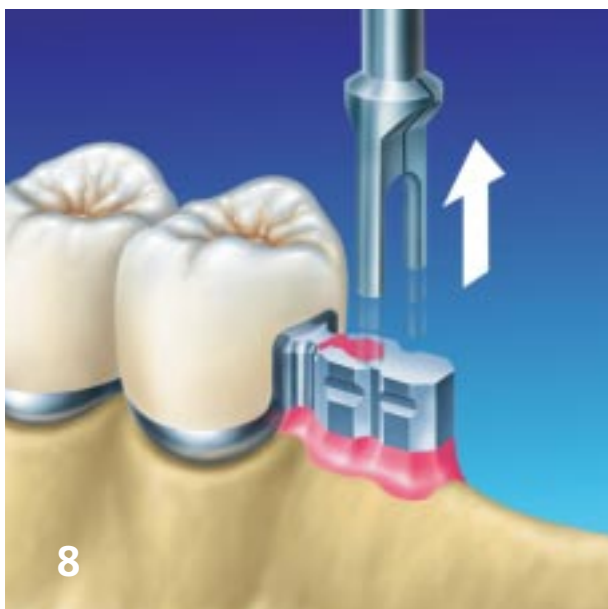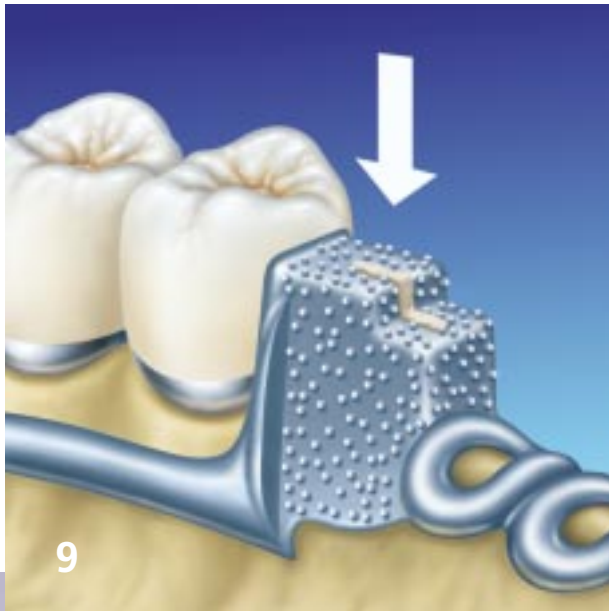
**Fig. 9:**

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
*Cover the housing with a thin layer of adhesive (e.g. Masterbond oder-no. 8300) and position the partial.*

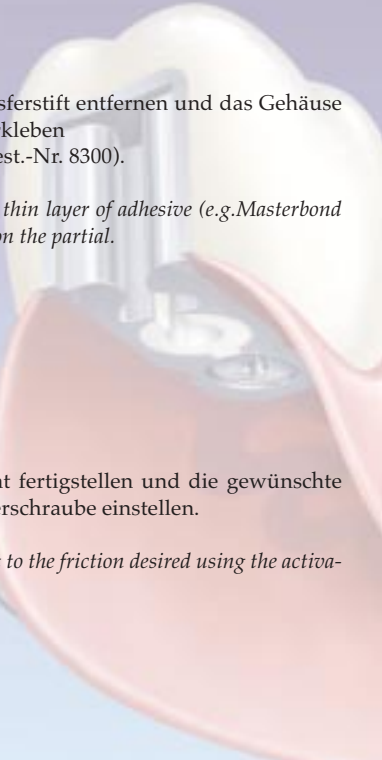**Fig. 10:**

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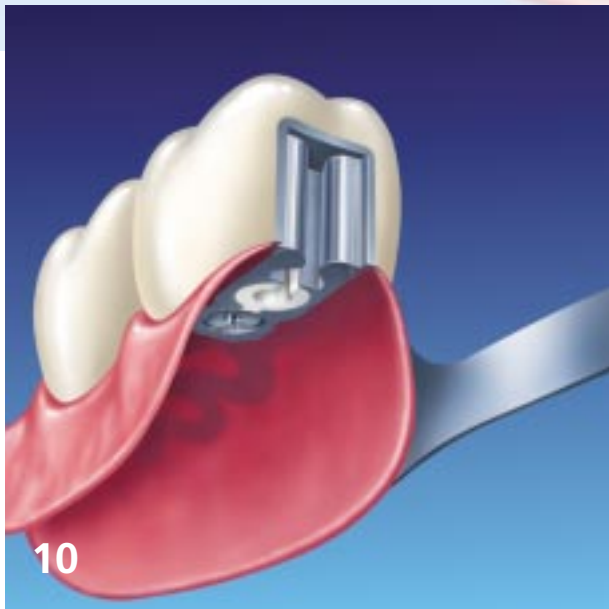*Finish as usual and adjust to the friction desired using the activating (deactivating) screw.*

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Servo-Dental GmbH & Co. KG