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SD base profiles for SD Ball anchors

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
Step 1:

A wax sheet is layed over the jaw ridge and holes are cut into the wax.

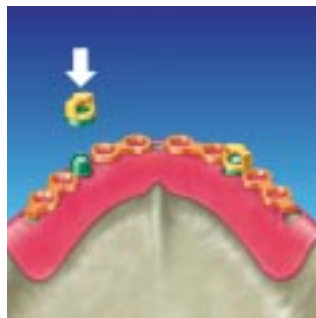Fix positioning ring over the SD Ball anchor. Clip the SD-ring retentions together and fix with super glue or use the mono block. If needed separate into single units.

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
Step 2:


The base profiles are fixed over the positioning rings and the retention elements are layed as shown.

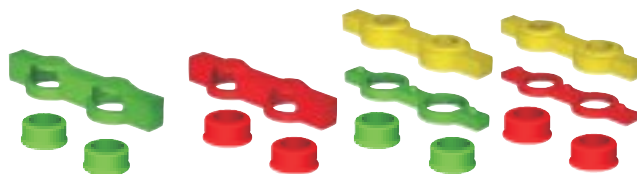The elements are then connected with a cold curing resin.

Introduction:

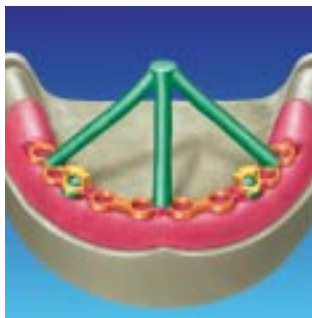The SD Base profile retention system (ring retentions) is like a building set. An easy way of varying, making metal structures for total prosthetics and on implants, without having to duplicate the original model.

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
Step 3:

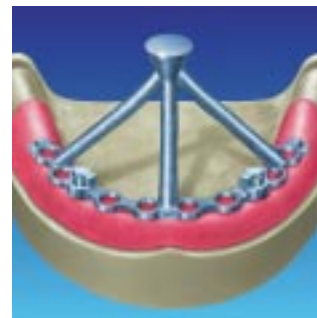After completing the build-up adapt sprues. It is easy to remove the structure from the model. Now invest and cast.

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Step 4:

After devesting and sandblasting, check the fit using the black female.

This black working aid is used until the prosthesis is finished.

After everything is finished remove the black working aid and insert the original female using the insertion tool. There are four different degrees of friction to choose from.

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Servo-Dental GmbH & Co. KG